


# INSTRUBAC 20L

## Protocole de traitement de l'Instrubac

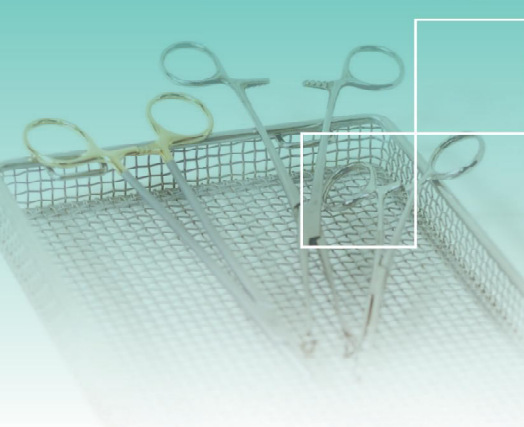

Bac de trempage non autoclavable



### NETTOYAGE

L'Instrubacs 20L est conçu pour être **lavés manuellement** avec un détergent/désinfectant. Se reporter au protocole interne de l'établissement.

### STOCKAGE

Les bacs non traités doivent être stockés dans leur emballage d'origine. Un second emballage est conseillé, par exemple, une boîte cartonnée. Les bacs doivent être **stockés dans un endroit sec** (minimum supérieur à 4°C) **sans contact direct** avec le sol.

### RÉFÉRENCES

- |  |               |
|--|---------------|
| ① Instrubac 20L sans robinet .....     | Réf. 404 006  |
| ② Instrubac 20L robinet à droite.....  | Réf. 404 023  |
| Instrubac 20L robinet au centre.....   | Réf. 404 023C |
| ③ Instrubac 20L robinet à gauche.....  | Réf. 404 023G |
| Instrubac 20L couvercle coiffant ..... | Réf. 404 023D |



### RECOMMANDATIONS :

- Les bacs ne sont pas autoclavables. Ils ne sont pas conçus pour un nettoyage en laveuse automatique.
- Une inspection visuelle est nécessaire pour évaluer l'usure du bac dû à son utilisation. S'il est endommagé, ne pas utiliser le bac et le signaler.

Dispositif Médical de classe I.

Les Laboratoires Anios déclinent toutes responsabilités en cas de non-respect des consignes de traitement des Instrubacs.

# INSTRUBAC 20L

## Cleaning protocol of Instrubac



Non-autoclavable  
soaking  
tank



### CLEANING

Instrubacs 20L is designed so as to be **manually cleaned** with detergent/desinfectant. See protocol established in your department.

### STORAGE

Untreated tanks must be stored in their original packaging. Secondary packaging is recommended, for example, a cardboard box. The tanks must be stored **in a dry place** (minimum 4°C) **without direct contact** with the ground.

### REFERENCES

- 1 Instrubac 20L without tap.....Ref. 404 006
- 2 Instrubac 20L tap on the right.....Ref. 404 023  
Instrubac 20L tap in the center.....Ref. 404 023G
- 3 Instrubac 20L tap on the left.....Ref. 404 023G  
Instrubac 20L covering lid.....Ref. 404 023D



### RECOMMANDATIONS :

- The tanks are not autoclavable, they are not designed for a cleaning in automatic washer machine.
- A visual inspection is necessary to evaluate wear to the tank due to its use. If it is damaged, do not use the tank and report it.

Class I medical device.

Laboratoires Anios disclaim all liability in case of non-compliance with the cleaning protocol of Instrubacs.