

ANIOS' CLEAN EXCEL D



Detergent
Instrumentation
pre-disinfectant

PRODUCT
TO BE DILUTED
25 ml > 5L



0,5%
5 min.



INDICATIONS

Cleaning and disinfection of medical devices: surgical and medical instruments, temperature sensitive instruments and endoscopy equipment, before sterilization.

PROTOCOL OF USE

INSTRUMENTATION



1

0,5% dilution: Pour a dose of 25 ml for 5 liters of cold or lukewarm water.



2

Completely submerge the medical device.
Soaking time: **5 minutes minimum.**
Brush, swab if there are visible stains.



3

Thoroughly rinse the treated medical devices with mains water (of good microbiological quality) remove all traces of product and organic soiling.

- ▶ **Heat sensitive material:**
Take the next step
(see internal procedure of the establishment)
- ▶ **Autoclavable material:**
Dry thoroughly with a clean drape and go to the next step
(see internal procedure of the establishment)

Dangerous - follow the precautions of use