

# OPASTER'ANIOS

## BANDETTES DE CONTROLE - TESTING STRIPS

### INDICATIONS

50 bandelettes réactives spécifiques pour le contrôle de la concentration d'orthophtaldéhyde dans la solution d'OPASTER'ANIOS.

### MODE D'EMPLOI

Tremper la bandelette pendant une seconde.

Éliminer l'excès de solution par une légère agitation de la bandelette.

Lire le résultat 60 secondes après trempage. Respecter le délai avant interprétation.

### CONDITIONS DE STOCKAGE

Conserver le flacon fermé entre +4°C et +30°C dans un endroit sec pendant 24 mois. Conserver à l'abri de la lumière et de l'humidité.

Si ces conditions sont respectées, les bandelettes sont utilisables jusqu'à 90 jours après ouverture.

### INDICATIONS

50 specific reactive strips for testing the concentration of orthophtaldehyde in the solution OPASTER'ANIOS.

### INSTRUCTIONS FOR USE

Soak the strip for one second.

Shake off excess liquid.


Read the result 60 seconds after steeping. Respect this time before reading.

### STORAGE CONDITIONS

Keep the flask firmly closed at a temperature between +4°C and +30°C, in a dry area for 24 months. Store away from light and moisture.


If these conditions are respected, testing strips are usable until 90 days after opening.

Rose uniforme  
**CONFORME**

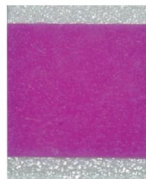

Pink uniform  
**CONFORM**

Rose non uniforme  
**NON CONFORME**

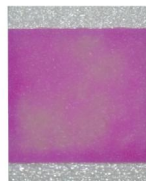

Pink not uniform  
**NO CONFORM**



Laboratoires

**ANIOS**

Le professionnel de la désinfection

Pavé du Moulin

59260 Lille-Hellemmes - France

Tel. +33 3 20 67 67 67 - Fax: +33 3 20 67 67 68

www.anios.com

FG100110-151112