Technical Data Sheet

Plum Eyewash pH Neutral 4.9% Phosphate

1. Identification of the product and the company

1.1. Product identifier and intended use.

Trade Name: Plum Eyewash pH Neutral 4.9% Phosphate

Intended Use: The products are intended to be used as first aid products for eye irrigation after accidents involving

acids and alkali.

Expiry date: Printed on the bottle label.

1.2. Details of the supplier of the data sheet

Manufacturer

Company: Plum A/S

Address: Frederik Plums Vej 2

 Zip code:
 5610

 City:
 Assens

 Country
 DENMARK

 E-mail
 info@plum.eu

 Phone
 +45 64712112

 Fax
 +45 64712125

1.3 Legislative information

General information:

As the products are class 1s sterile medical devices according to Council Directive 93/42/EEC of 14

June 1993 concerning medical devices, they are certified by Notified Body, Presafe.

No Safety Data Sheet is required for medical devices.

The products shall not be classified as hazardous according to the classification and labeling rules for substance and mixtures.

Plum A/S is certified according to ISO 13485:2016 Medical devices – Quality management systems – Requirements for regulatory purposes.

1.4 Instruction for use:

Mono eye cup (200 and 500ml) / Shower eye cup (1000 ml)



DUO eye cup (500 ml)



- Always wash with an ordinary cycwash solution after using Plum Eyewash pH Neutral 4.9% Phosphate!
- Do not reuse.
- Do not use if damaged.
- Recommended temperature when used







Technical Data Sheet

Plum Eyewash pH Neutral 4.9% Phosphate

2. General information about the product

Material requirements:

The materials in these products have been judged biocompatible according the requirements of ISO 10993-1:2009 Biological evaluation of medical devices – Part 1: Evaluation and testing within a risk management process.

Bottle, label and eye cup dimensions.

	200 ml bottle	500 ml bottle	1000 ml bottle
Bottle material	Polyethylene without additives	Polyethylene without additives	Polyethylene without additives
The bottle contains [ml]	201-207	505-512	1005-1020
Bottom Ø [mm]	58.5	78	90
Height [mm]	129.6	183	229

3. Side effects and malfunction

Side effects and malfunction:

The products are designed as a first aid product for eye irrigation. If side effects or malfunctions are experienced, please report these to Plum A/S, <u>info@plum.eu</u> or phone +45 6471 2112, so that necessary remedial measures can be initiated. Serious adverse effects should always be reported to a relevant and competent authority.

4. Storage

Storage:

5°C 1 40°C

Shelf life: 42 months from manufacturing date.

Installation: Please refer to the pictogramm on the label:

THIS SIDE UP

Mounting stations and boxes: Please visit https://plum.eu/products/firstaid/eye-wash

5. Other information

Version history and indication of changes

Version	Revision date	Responsible	Changes
1.0	19-11-2019	CL	New TDS

Other information Additional information can be obtained by contacting info@plum.eu or by telephone +45 6471 2112.

TDS prepared by:

Company: Plum A/S

Address: Frederik Plums Vej 2

 Zip code:
 5610

 City
 Assens

 Country:
 DENMARK

 E-mail:
 info@plum.eu

 Phone:
 +45 64712112

 Fax:
 +45 64712125

Document Language: English