

# INSTRUBAC 15L

## Protocole de traitement de l'Instrubac



### NETTOYAGE

Les Instrubacs 15L sont conçus pour être **lavés manuellement** avec un détergent/désinfectant. Se reporter au protocole interne de l'établissement.

### STOCKAGE

Les bacs non traités doivent être stockés dans leur emballage d'origine. Un second emballage est conseillé, par exemple, une boîte cartonnée.

Les bacs doivent être **stockés dans un endroit sec** (minimum supérieur à 4°C) **sans contact direct** avec le sol.

### RÉFÉRENCES

- Bac simple sans robinet.....Réf. 404 450
- ① Bac double sans robinet.....Réf. 404 451
- Bac simple avec robinet.....Réf. 404 024
- Bac double avec robinet.....Réf. 404 025



### RECOMMANDATIONS :


- Les bacs ne sont pas autoclavables. Ils ne sont pas conçus pour un nettoyage en laveuse automatique.
- Une inspection visuelle est nécessaire pour évaluer l'usure du bac dû à son utilisation. S'il est endommagé, ne pas utiliser le bac et le signaler.

Dispositif Médical de classe I.


Les Laboratoires Anios déclinent toutes responsabilités en cas de non-respect des consignes de traitement des Instrubacs.

# INSTRUBAC 15L

## Cleaning protocol of Instrubac

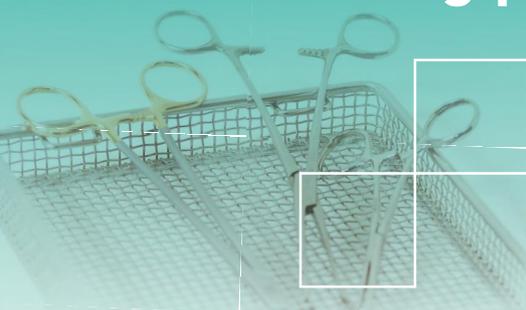

Non-autoclavable  
soaking  
tank

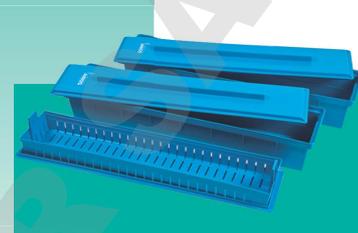

### CLEANING

Instrubac 15L are designed so as to be **manually cleaned** with detergent/desinfectant. See protocol established in your department.

### STORAGE

Untreated tanks must be stored in their original packaging. Secondary packaging is recommended, for example, a cardboard box. The tanks must be stored **in a dry place** (minimum 4°C) **without direct contact** with the ground.

### REFERENCES

- Simple box without tap.....Ref. 404 450
- ① Double box without tap.....Ref. 404 451
- Simple box with tap.....Ref. 404 024
- Double box with tap.....Ref. 404 025



### RECOMMANDATIONS :

- The tanks are not autoclavable. They are not designed for a cleaning in automatic washer machine.
- A visual inspection is necessary to evaluate wear to the tank due to its use. If it is damaged, do not use the tank and report it.

Class I medical device.

Laboratoires Anios disclaim all liability in case of non-compliance with the cleaning protocol of Instrubacs.