

INSTRUBAC 40L et 60L

Protocole de traitement de l'Instrubac



Bac de trempage non autoclavable



NETTOYAGE

Les Instrubacs 40L et 60L sont conçus pour être **lavés manuellement** avec un détergent/désinfectant. Se reporter au protocole interne de l'établissement.

STOCKAGE

Les bacs non traités doivent être stockés dans leur emballage d'origine. Un second emballage est conseillé, par exemple, une boîte cartonnée. Les bacs doivent être **stockés dans un endroit sec** (minimum supérieur à 4°C) **sans contact direct** avec le sol.

RÉFÉRENCES

- 1 Instrubacs 40L sans robinet.....Réf. 406 012
- 2 Instrubacs 40L avec robinet sur petit côté.....Réf. 404 412
Instrubacs 40L avec robinet sur grand côté.....Réf. 404 412A
- 3 Instrubacs 60L sans robinet.....Réf. 406 013
- 4 Instrubacs 60L avec robinet sur petit côté.....Réf. 404 413



RECOMMANDATIONS :

- Les bacs ne sont pas autoclavables. Ils ne sont pas conçus pour un nettoyage en laveuse automatique.
- Une inspection visuelle est nécessaire pour évaluer l'usure du bac dû à son utilisation. S'il est endommagé, ne pas utiliser le bac et le signaler.

Dispositif Médical de classe I.

Les Laboratoires Anios déclinent toutes responsabilités en cas de non-respect des consignes de traitement des Instrubacs.

INSTRUBAC 40L et 60L

Cleaning protocol of Instrubac



Non-autoclavable
soaking
tank



CLEANING

The Instrubacs 40L and 60L are designed so as to be **manually cleaned** with detergent/desinfectant. See protocol established in your department.

STORAGE

Untreated tanks must be stored in their original packaging.
Secondary packaging is recommended, for example, a cardboard box.
The tanks must be stored **in a dry place** (minimum 4°C) **without direct contact** with the ground.

REFERENCES

- 1 Instrubacs 40L without tap.....Ref. 406 012
- 2 Instrubacs 40L with tap on the short side.....Ref. 404 412
Instrubacs 40L with tap on the long sideRef. 404 412A
- 3 Instrubacs 60L without tap.....Ref. 406 013
- 4 Instrubacs 60L with tap on the short side.....Ref. 404 413



RECOMMANDATIONS :

- The tanks are not autoclavable. They are not designed for a cleaning in automatic washer machine.
- A visual inspection is necessary to evaluate wear to the tank due to its use. If it is damaged, do not use the tank and report it.

Class I medical device.

Laboratoires Anios disclaim all liability in case of non-compliance with the cleaning protocol of Instrubacs.