75/387027/1011

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75/387027/1011

## Aqua B. Braun

Irrigation solution, sterile and endotoxin-free

**Composition**  
1000 ml solution contain:  
Water for Injection 1000 ml

**Indications**  
For irrigation and cleansing during surgical intervention

For irrigation of wounds and burns

For moistening wound tamponades, cloths and dressings  
For rinsing and cleansing of bladder catheters  
For irrigation and cleansing in stoma care

For rinsing and cleansing of instruments and equipment  
For filling respiratory air humidifiers (the use of container equal or higher than 100 ml is recommended)

For mechanical irrigation of the eyes (the use of 30 ml container is recommended)

**Dosage**  
The amount of fluid depends on the clinical condition.

**Overdosage**  
Overdosage during wound irrigation and intra-operative application can lead to absorption of irrigation fluid, that can lead to hypervolaemia in the form of hypotonic

hyperhydration (hyponatraemia). Headache, nausea, restlessness and disorientation are symptoms

of hyperhydration. In severe cases coma-like states can occur.  
**Countermeasures in cases of overdosage**  
Discontinuation of irrigation, correction of the water and electrolyte balance depending on the particular clinical situation.

**Warnings and precautions**  
Do not use for infusion.

Only use the solution if the seal is undamaged and the solution is clear.  
Opened containers must not be saved, quantities left over must be discarded.

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.  
Store out of reach of children.

Aqua B. Braun must not be used for intratecal irrigation procedures.

**Adverse side effects**  
None are known.

**Expiry date**  
The solution must not be used after the expiry date given on the label.

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