



Guest Medical

Dedicated to Care, Quality and Service

www.guest-medical.co.uk

Blood Spillage Disinfection

Where blood or blood stained body fluids have been spilled, **DoH recommendations** are to use **10,000 ppm** available chlorine either from granules or a solution.



The Multi-Use Biohazard Spills Kits contain Haz-Tab Granules and Haz-Tab Tablets.

Blood Spillage Disinfection

How do we know Haz-Tab Products work?

- ✓ BS:EN 1276 A suspension test for bacteria
- ✓ BS:EN 1650 A suspension test for fungi
- ✓ BS:EN 13727 A suspension test for vegetative bacteria
- ✓ BS:EN 13624 A suspension test for fungal spores
- ✓ BS:EN 13697 A surface test for bacteria and/or fungi
- ✓ BS:EN 13704 A suspension test for bacterial spores
- ✓ BS:EN 14348 A suspension test for Mycobacteria
- ✓ BS:EN 14476 A suspension test for viruses

Efficacy has been proven through independent testing at UKAS accredited laboratories


How to Use Biohazard Spill Kit

Before you start:



- ✓ Ensure that you know how to use the kit correctly by following the instruction card

Guest Medical www.guest-medical.co.uk

BIOHAZARD SPILL KIT

BEFORE YOU START:
Always wear gloves and an apron.  Wherever possible ensure good ventilation. 

NEVER use on chemical spillages


- 1 Sprinkle Haz-Tab Granules over spill until all moisture is absorbed. Leave for no more than 2 minutes. 
- 2 Set up the waste bag close to the spill. **FOR SPLASHES AND VERTICAL SURFACES:** Place 4 Haz-Tab tablets in the diluter and fill to the line with cold tap water. Allow to dissolve. 
- 3 Collect spill and granules mixture using scoop and scraper. Discard all into the yellow disposal bag. 
- 4 When tablets have dissolved gently invert diluter several times to mix solution. **DO NOT SHAKE DILUTER.** Use with the paper towels to wipe over area of spill and remove splashes on vertical surfaces. 
- 5 Discard remaining solution by flushing down the sluice with plenty of cold water. Always flush sluice before and after discarding solution. **REPLACE CAP & KEEP THE DILUTER FOR NEXT USE.** 
- 6 Place all used materials, gloves last, into the disposal bag. Secure with the backing strip as a tie. Dispose of bag in accordance with your local clinical waste policy. **NOW WASH YOUR HANDS.** 

Manufactured by **Guest Medical** Aylesford, Kent
Order Codes: H8615 Midi Kit ME20 GSW Telephone: +44 (0) 1622 791895
H8616 Large Kit United Kingdom Email: enquiries@guest-medical.co.uk

How to Use Biohazard Spill Kit

Before you start:

- ✓ Always wear protective gloves and apron provided

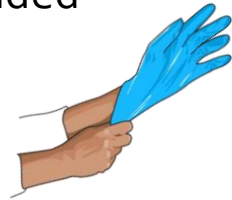

- ✓ Wherever possible, improve room ventilation



- ✓ Always check expiry dates on product containers



How to Use Haz-Tab Granules

For spillages on flat surfaces:



At arms length, shake the granules over the entire spill



Keep shaking until the spill is completely covered and starting to be absorbed




After 2 minutes, collect granules and spill mixture and discard into yellow waste bag

How to Use Haz-Tab Tablets

Fill diluter with cold water to the fill line



Always check the cold tap is running cold

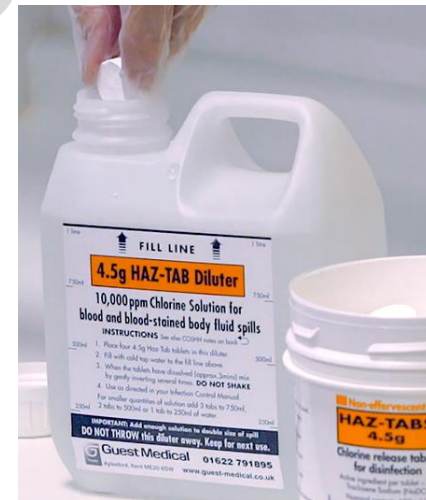

Add four tablets to 1 litre of cold water for 10,000ppm available chlorine solution

Leave to dissolve for 2 minutes then **gently** invert diluter to mix.



Use the Haz-Tab solution with paper towels to wipe over area of spill or to remove splashes on vertical surfaces.

How to Use Haz-Tab Tablets


Discarding used solution:



Flush the sink or sluice with cold water first



Pour away the solution

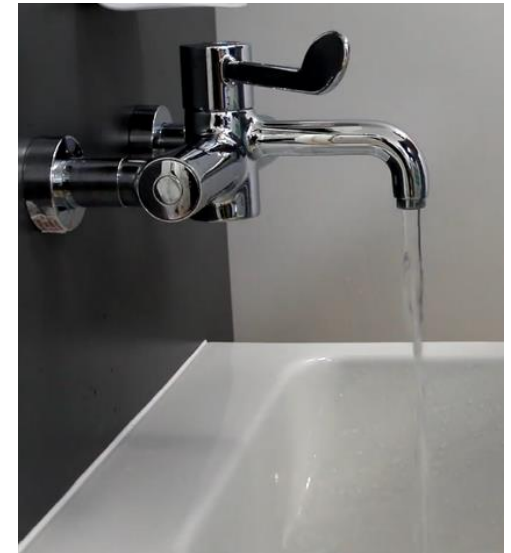

Flush the sink or sluice with cold water again

How to Use Biohazard Spill Kit

Once the procedure is complete:

- ✓ Place all used materials, gloves last into the yellow waste bag



- ✓ Always tie bag and discard in accordance with local clinical waste policy




- ✓ Wash hands thoroughly after using kit

Remember



✗ **Never** shake the container to mix


✗ **Never** mix products with any other cleaning agent or any other chemical

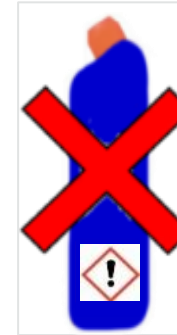

✗ **Never** use Haz-Tab Solutions in a spray bottle



✗ **Never** make up solutions with hot water



Any Questions?

Guest Medical - Dedicated to Care, Quality and Service

www.guest-medical.co.uk