McFARLAND BARIUM SULPHATE STANDARD

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DESCRIPTION

McFarland standards are used as turbidity standards in the preparation of suspensions of microorganisms and has particular application in the preparation of bacterial inocula for performing antimicrobial susceptibility testing.

PRINCIPLE

Turbidity standards are prepared by mixing chemicals that precipitate to form a solution of reproducible turbidity.

McFarland standards are prepared by adding sulphuric acid to an aqueous solution of barium chloride, which results in the formation of a suspended barium sulphate precipitate.

For each McFarland standard in table 1 is reported the correspondent density expressed in cells/ml. Bacterial concentration depends on microorganisms size. The mentioned values in table 1 represent average values of concentration valid for bacteria. For yeast, which are larger in size, these numbers should be divided by about 30.

PROCEDURE

Vigorously agitate the turbidity standard on a mechanical vortex mixer just before use.

Using adequate light, compare the turbidity of a bacterial suspension to the turbidity standard.

Alternatively, use the turbidity standard to calibrate a electrometric turbidimeter.

RESULTS INTERPRETATION

McFarland standards will enable the preparation of standardized inocula for use in the performance of standardized antimicrobial susceptibility testing procedures.

Tabella / Table 1.

McFarland Standard	Densità (cellule/ml) / Density (cells/ml)
0.5	1.5×10^8
1.0	3.0×10^8
2.0	6.0×10^8
3.0	9.0×10^8
4.0	12.0×10^8

BIBLIOGRAFIA / BIBLIOGRAPHY

1. Mc Farland, 1907. J.Am.Med.Assoc.49:1176.
2. Patricia M. Tille. 2014. Bailey & Scott's Diagnostic Microbiology, 13th edition by Mosby, Inc., an affiliate of Elsevier Inc.
3. CLSI M7-A9, 2012. Methods for dilution antimicrobial susceptibility test for bacteria that grow aerobically.
4. CLSI M11-A7, 2007. Methods for dilution antimicrobial susceptibility testing of anaerobic bacteria.

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Prodotto / Product	REF	Σ
McFARLAND 0.5 BARIUM SULPHATE STANDARD	80400	1
McFARLAND 1.0 BARIUM SULPHATE STANDARD	80401	1
McFARLAND 2.0 BARIUM SULPHATE STANDARD	80402	1
McFARLAND 3.0 BARIUM SULPHATE STANDARD	80403	1
McFARLAND 4.0 BARIUM SULPHATE STANDARD	80404	1
McFARLAND STANDARD SET (McFARLAND 0.5, 1.0, 2.0, 3.0, 4.0)	80405	5

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