

Oosafe® 6 Well Dish with Straw Holder, Non-Treated Surface

Description (L x W x H) (mm)	Prod. No.	Qty/pc	Qty/cs
77.2 x 64.4 x 12.4	OOPW-SW02	1	240
77.2 x 64.4 x 12.4	OOPW-SW03	4	120

Intended use

General Intended Use: For the examination of specimens, including blood and tissue donations, derived from the human body.

Specific Intended Use: Aid in the culturing and observation of cells growth. Special conical flat bottom for better optical visualization of smaller cells.



Product compartments: 6

Dish Volumes:

- Well compartment 1: 0.4 ml
- Well compartment 2: 0.4 ml
- Well compartment 3: 0.4 ml
- Well compartment 4: 0.4 ml
- Well compartment 5: 0.4 ml
- Well compartment 6: 0.4 ml

Safety


Observe the expiry date and the storage condition on the label.
Do only use optically perfect and undamaged plasticware.

Handling

The package of Oosafe® 6 Well Dish with Straw Holder including single pack and multipack, is breathable material with no VOC release. After opening the sealing, the dishes can be directly put into use.

To culture the cells, fill the wells with medium, and place the dish in the incubator or in the heating unit prior to use. The recommended medium volume per well is 0,15 ml if used in conjunction with oil, otherwise 0.3 ml.

When the dish is covered by the lid, the air can still go in and out of the dish.

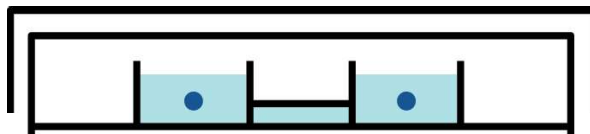

Discard after use.

Storage: Store at room temperature.