
Oosafe® 50 mm Dish, Thin bottom, Non-Treated Surface

Description (D, H) ² (mm)	Prod. No.	Qty/pc	Qty/cs
52.2, 11.2	OOPW-IC06	10	500

Intended Use

General Intended Use: For the examination of specimens, including blood and tissue donations, derived from the human body.

Specific Intended Use: Aid in examining the quality of a sample under microscope e.g. sperm, allows for better imaging under microscope at higher magnification.

Product compartments: 1

Dish Volumes:

Well compartment 1: 15 ml

Safety

Observe the expiry date and the storage condition on the label.
Do only use optically perfect and undamaged plasticware.


Handling

Prior to use, open the sealing and leave the dishes to off-gas before use (unless it is a single-pack dish).

Oosafe® 50 mm Dish, Label Area Grip packages may be off-gassed by partially opening the sleeve or taking the dishes entirely out of the packaging. In either case, the opened sleeves or dishes must be kept in a sterile environment at room temperature and humidity. The dishes should remain covered with the lids until they are loaded with the sample and medium.

Place small droplets on the bottom of the dish. Recommended volume per drop is 5 - 50 µl according to the use purpose. Overlay the drop with oil. The recommended volume is 5 ml.

When the dish is covered by the lid, the air can still go in and out of the dish.

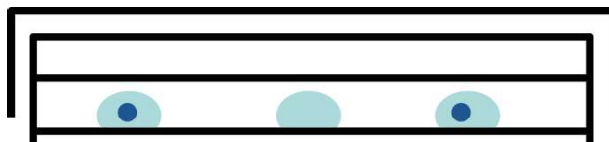

Discard after use.

Storage: Store at room temperature.