
Oosafe® Center Well Dish with Two Compartments, Label Area Grip, Non-Treated Surface

Description (D, H) (mm)	Prod. No.	Qty/pc	Qty/cs
59.0, 15.0	OOPW-CW05	10	500

Intended use

General Intended Use: For the examination of specimens, including blood and tissue donations, derived from the human body.

Specific Intended Use: Aid in 3 culturing groups of cell culture with different chamber sizes. As well as examining the cells and observing the growth of the cells under the same conditions.

Product compartments: 3

Dish Volumes:

Well compartment 1: 3 ml

Well compartment 2: 5 ml

Well compartment 3: 5 ml



Safety

Observe the expiry date and the storage condition on the label.

Do only use optically perfect and undamaged plasticware.


Handling

Prior to use, open the sealing and leave the dishes to off-gas before use (unless it is a single-pack dish).

Oosafe® Center Well Dish with Two Compartments, Label Area Grip packages may be off-gassed by partially opening the sleeve or taking the dishes entirely out of the packaging. In either case, the opened sleeves or dishes must be kept in a sterile environment at room temperature and humidity. The dishes should remain covered with the lids until they are loaded with the sample and medium.

Fill the center well and the outer chambers with equilibrated medium according to your protocol. For well compartment 1 the recommended volume is 1 ml if used in conjunction with oil, otherwise 2 ml. For well compartment 2 and 3 the recommended volume is 2 ml if used in conjunction with oil, otherwise 4 ml.

When the dish is covered by the lid, the air can still go in and out of the dish.

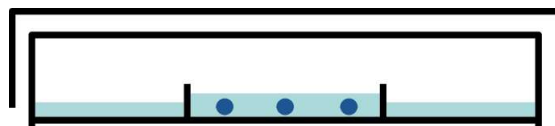

Discard after use.

Storage: Store at room temperature.