
Oosafe® Sperm Collection Cup, 80 ml, Non-Treated Surface

Description (volume) (ml)	Prod. No.	Qty/pc	Qty/cs
80 ml	OOPW-SC01	1	70

Intended Use

General Intended Use: For the examination of specimens, including blood and tissue donations, derived from the human body.

Specific Intended Use: Aid in collection of sperm and examination of the quality of sperm, in terms of quantitative and qualitative measurements e.g. pH, color, liquidity.

Product compartments: 1

Dish Volumes:

Cup compartment 1: 80 ml



Safety


Observe the expiry date and the storage condition on the label. Do only use optically perfect and undamaged plasticware.

Handling

The packaging of Oosafe® Sperm Collection Cup is breathable material with no VOC release. After opening the sealing, the cup can be directly put into use.

Screw the cap properly to avoid leaking after collecting the sperm.

The cup and the cap are made of Polypropylene. Do not squeeze the cup and the cap too much, otherwise leaking may happen by reason of the cap not being closed properly.

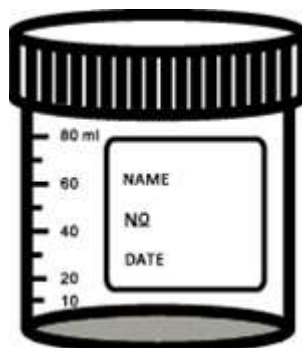

Discard after use.

Storage: Store at room temperature.