


BD Vacutainer® SST™ II Advance Tube

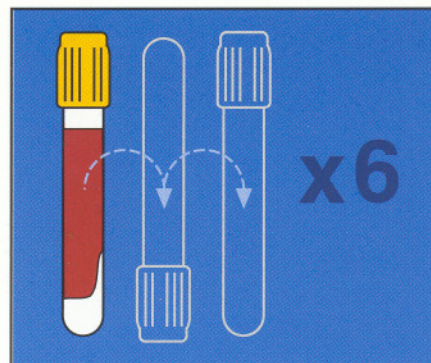


x6




1

30 min

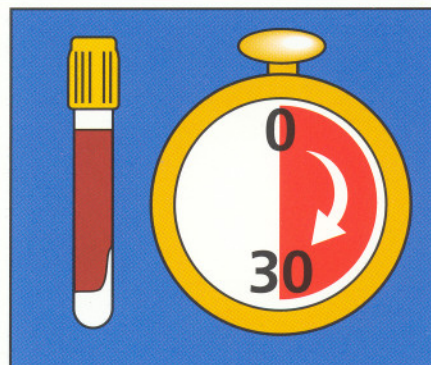


2

10 min
1300-2000g
20-25°C

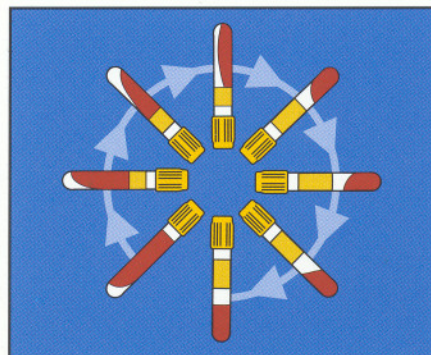

3

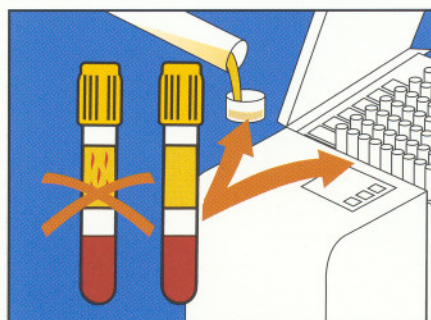


4

1. NCCLS recommendation: centrifugation within 2 hrs of collection. Recommendation NCCLS: centrifugation dans les 2 h après le prélèvement. NCCLS Empfehlung: Zentrifugation innerhalb 2 std nach Entnahme. Raccomandazioni NCCLS: centrifugare entro 02:00 h dal prelievo. NCCLS aanbeveling: centrifugatie binnen 2 u na afname. NCCLS rekommenderar centrifugering inom 2 timmar. Recomendación NCCLS: centrifugar dentro de las 2 h posteriores a la extracción. NCCLS Document H18-A2: Procedures for the Handling and Processing of Blood Specimens, Second Edition, Approved Guideline; Wayne, PA, 1999.

BD Vacutainer® PST™ II Plus Tube



x8-10

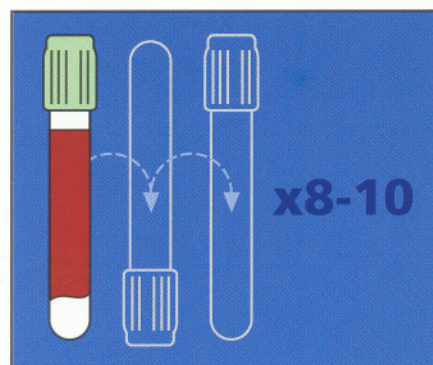


1


- No time for clotting.
- Pas de délai avant centrifugation.
- Gerinnungszeit muss nicht abgewartet werden.
- Nessun tempo di attesa per la formazione del coagulo.
- Geen stollingstijd.
- OBS! Vänta ej på koagulation.
- Se puede centrifugar inmediatamente.

2

10 min
1300-2000g
20-25°C

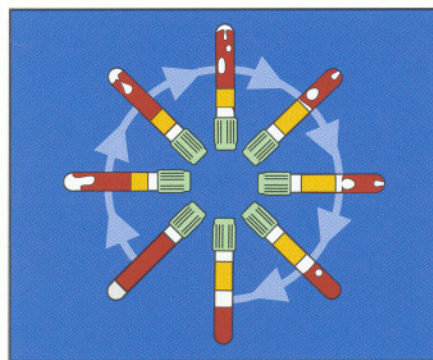

3

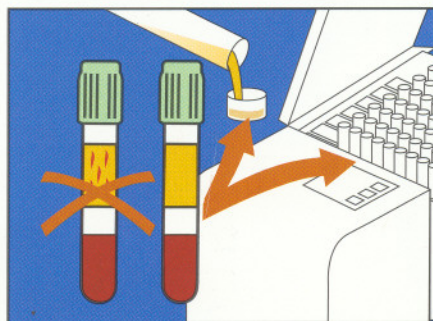

4

1. NCCLS recommendation: centrifugation within 2 hrs of collection. Recommendation NCCLS: centrifugation dans les 2 h après le prélèvement. NCCLS Empfehlung: Zentrifugation innerhalb 2 std nach Entnahme. Raccomandazioni NCCLS: centrifugare entro 02:00 h dal prelievo. NCCLS aanbeveling: centrifugatie binnen 2 u na afname. NCCLS rekommenderar centrifugering inom 2 timmar. Recomendación NCCLS: centrifugar dentro de las 2 h posteriores a la extracción. NCCLS Document H18-A2: Procedures for the Handling and Processing of Blood Specimens, Second Edition, Approved Guideline; Wayne, PA, 1999.