

pfmmedical

Quality and Experience

Surgical Instruments

- › Surgical Scalpels
- › Surgical Blades and Handles
- › Disposable Blade Removers



FEATHER® - World class in quality and consistency for cutting instruments

www.pfmmedical.com




FEATHER® Surgical Scalpels

General Benefits

Due to its optimised micrometer-accurate precision grinding, the high-quality stainless steel blade offers outstanding quality and performance. A double-honing process ensures ultra-sharp cutting edges of superb durability.

General Handling

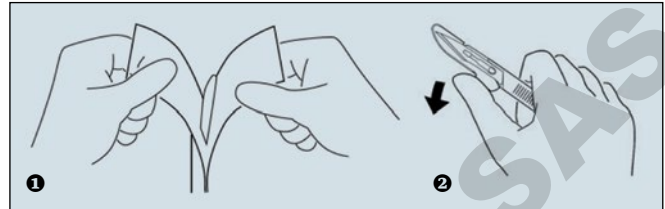

- ❶ **Open the aluminium foil packaging as illustrated:**
Pull the two flaps apart.
- ❷ **Removal of the blade protection:**
Press gently on the snap tab as illustrated.

Standard Scalpels

The disposable Standard Scalpel with high-quality stainless steel blade is suitable for all surgical applications.

View



Benefits

Unsurpassed quality and performance

Due to its optimised micrometer-accurate precision grinding, the high-quality stainless steel blade of the standard scalpel offers outstanding quality and performance. A double-honing process ensures ultra-sharp cutting edges of superb durability.

Ordering Information

Standard Scalpels

REF	Figuration	PU
200130010	10	20
200130011	11	20
200130012	12	20
205023003	12 D	20
205143003	14	20
200130015	15	20
205353003	15 C	20
200130020	20	20
200130021	21	20
200130022	22	20
200130023	23	20
200130024	24	20
200130025	25	20

Scalpels, long

The scalpel with a plastic handle which is twice the length (23 cm) of a standard model makes it more convenient to use in deep operations.

View



Ordering Information

Scalpels, long

REF	Figuration	PU
200210010	10	20
200210011	11	20
200210015	15	20
200210020	20	20
200210021	21	20
200210022	22	20
200210023	23	20

Scalpels, mini

The mini scalpel with its 18 cm handle is ideally suited for all surgical procedures with narrow operating fields.

View



Ordering Information

Scalpels, mini

REF	Figuration	PU
200170011	11	20
200170014	14	20
200170015	15	20

Safety Scalpels

The FEATHER® Safeshield™ disposable scalpel is designed for ultimate safety during surgical procedures.

View



Benefits

Optimum safety

The scalpel ensures maximum safety due to its Safeshield™ blade protection which can be closed with one hand.

- ▶ Balanced design
- ▶ Single-handed opening and closing of the blade protection
- ▶ Outstanding durability
- ▶ Individual sterile packaging
- ▶ Color-coded scalpels for easy blade shape identification

Ordering Information

Safety Scalpels

REF	Figuration	PU
200140010	10	10
200140011	11	10
200140014	14	10
200140015	15	10
200140115	15c	10
200140020	20	10
200140021	21	10
200140022	22	10
200140023	23	10
200140024	24	10
200140025	25	10

FEATHER® Surgical Blades and Handles

General Benefits

The surgical stainless steel blades made from the highest quality hardened stainless steel offer outstanding quality and performance due to their optimised micrometer-accurate precision grinding. A double-honing process ensures ultra-sharp cutting edges of superb durability.


Easy to use

The blades are ready for use immediately after removal from the packaging

Stainless Steel Blades





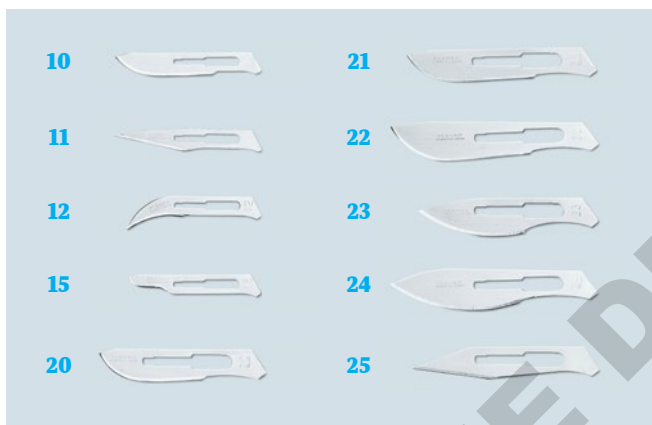




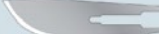

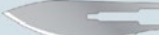

FEATHER® Stainless Steel Blades made from the highest quality hardened stainless steel are suitable for all surgical applications.

View



Ordering Information

Stainless Steel Blades

REF	Figuration	PU
201000010	10 	100
201000011	11 	100
201000012	12 	100
205020003	12 D 	100
205140003	14 	100
201000015	15 	100
205350003	15 C 	100
201000020	20 	100
201000021	21 	100
201000022	22 	100
201000023	23 	100
201000024	24 	100
201000025	25 	100

Carbon Steel Blades










FEATHER® Carbon Steel Blades with an anti-rust coating are suitable for all surgical applications.

View



Ordering Information

Carbon Steel Blades

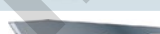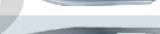REF	Figuration	PU
201500010	10 	100
201500011	11 	100
201500012	12 	100
201500015	15 	100
201500020	20 	100
201500021	21 	100
201500022	22 	100
201500023	23 	100
201500024	24 	100

Handles

FEATHER® Handles are suitable for carbon steel and stainless steel blades.

View



Ordering Information

Handles

REF	Version	Figuration	Size	Material	PU
202000003	Standard	10 - 15	3	Stainless steel	10
202000004	Standard	20 - 25	4	Stainless steel	10
202000007	Standard	10 - 15	7	Stainless steel	10
202010003	Long	10 - 15	3L	Stainless steel	10
202010004	Long	20 - 25	4L	Stainless steel	10
208800077	Standard	10 - 15	3DLC	Diamond-like carbon film coating	1
208800078	Standard	20 - 25	4DLC	Diamond-like carbon film coating	1
208800079	Standard	10 - 15	7DLC	Diamond-like carbon film coating	1
208800080	Round	10 - 15	3T	Titanium	1

Disposable Blade Removers

View




Standard Blade Remover



Palm-size Blade Remover

Handling

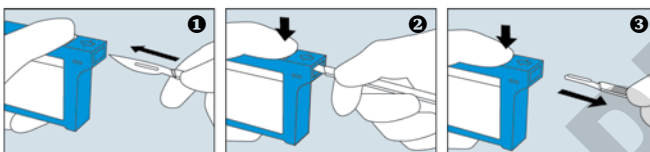

Benefits

Ease of use

The FEATHER® Blade Remover permits the safe and simple removal of used disposable blades from any kind and size of scalpel handle. It makes blade changes easy and ensures increased safety and hygiene because direct contact with the blade is avoided. Easy handling in three steps.

Safe and hygienic

No direct contact with blades provides the greatest possible protection of medical staff against infection.

Disposable product

Once the cost-effective blade remover is filled with used blades, it can simply be disposed of.

Instructions for use

- ① Make sure that the correct side of the blade is pointing upwards; then insert the blade into the slot of the blade remover until it hits the stop.
- ② Gently press the button on the top of the blade remover.
- ③ Remove the handle. The removed blade drops safely into the enclosed blade remover.

Ordering Information

Disposable Blade removers

REF	Version	PU
201090000	Disposable Blade Remover standard	1
201091000	Disposable Blade Remover palm-size	1

Contact

Should you have any questions our Customer Service & Sales Support team is happy to advise you.

service@pfmmmedical.com

T +49 (0)2236 9641-220

F +49 (0)2236 9641-51

www.pfmmmedical.com - Your information source for the products of pfm medical ag.

pfmmedical

pfm medical ag
Wankelstraße 60
50996 Köln, Germany

Exclusive Distributor
for Europe

Certified by
DIN EN ISO 13485

Manufacturer
FEATHER® Safety Razor Co. Ltd.



Registered Trademark in Japan
Products are subject to alterations