


INSTRUBAC 20L

Protocole de traitement de l'Instrubac

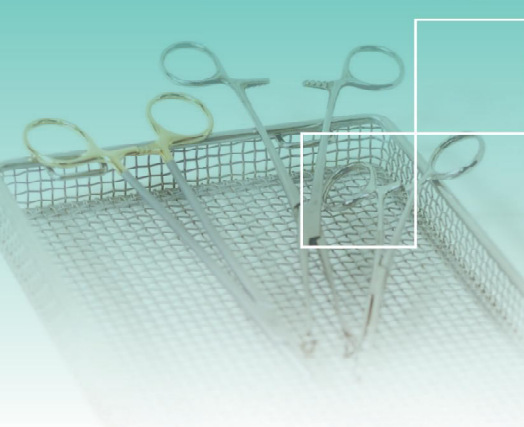

Bac de trempage non autoclavable



NETTOYAGE

L'Instrubacs 20L est conçu pour être **lavés manuellement** avec un détergent/désinfectant. Se reporter au protocole interne de l'établissement.

STOCKAGE

Les bacs non traités doivent être stockés dans leur emballage d'origine. Un second emballage est conseillé, par exemple, une boîte cartonnée. Les bacs doivent être **stockés dans un endroit sec** (minimum supérieur à 4°C) **sans contact direct** avec le sol.

RÉFÉRENCES

- | | |
|----------------------------------------|---------------|
| ① Instrubac 20L sans robinet | Réf. 404 006 |
| ② Instrubac 20L robinet à droite..... | Réf. 404 023 |
| Instrubac 20L robinet au centre..... | Réf. 404 023C |
| ③ Instrubac 20L robinet à gauche..... | Réf. 404 023G |
| Instrubac 20L couvercle coiffant | Réf. 404 023D |



RECOMMANDATIONS :


- Les bacs ne sont pas autoclavables. Ils ne sont pas conçus pour un nettoyage en laveuse automatique.
- Une inspection visuelle est nécessaire pour évaluer l'usure du bac dû à son utilisation. S'il est endommagé, ne pas utiliser le bac et le signaler.

Dispositif Médical de classe I.

Les Laboratoires Anios déclinent toutes responsabilités en cas de non-respect des consignes de traitement des Instrubacs.

INSTRUBAC 20L

Cleaning protocol of Instrubac

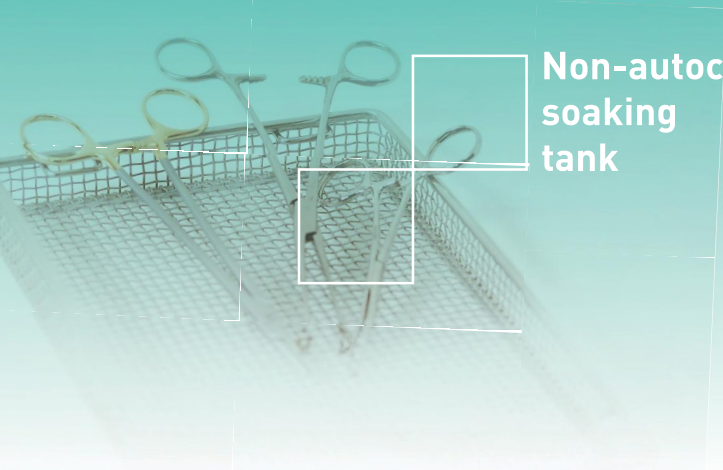

Non-autoclavable
soaking
tank



CLEANING

Instrubacs 20L is designed so as to be **manually cleaned** with detergent/desinfectant. See protocol established in your department.

STORAGE

Untreated tanks must be stored in their original packaging. Secondary packaging is recommended, for example, a cardboard box. The tanks must be stored **in a dry place** (minimum 4°C) **without direct contact** with the ground.

REFERENCES

- 1 Instrubac 20L without tap.....Ref. 404 006
- 2 Instrubac 20L tap on the right.....Ref. 404 023
Instrubac 20L tap in the center.....Ref. 404 023G
- 3 Instrubac 20L tap on the left.....Ref. 404 023G
Instrubac 20L covering lid.....Ref. 404 023D



RECOMMANDATIONS :

- The tanks are not autoclavable, they are not designed for a cleaning in automatic washer machine.
- A visual inspection is necessary to evaluate wear to the tank due to its use. If it is damaged, do not use the tank and report it.

Class I medical device.

Laboratoires Anios disclaim all liability in case of non-compliance with the cleaning protocol of Instrubacs.