



Reference: 4100

Technical Data Sheet

Product: LOWENSTEIN JENSEN MEDIUM

CEIVD

Specification

Culture and differentiation of Mycobacterium spp.

Presentation

Shelf Life Storage **Packaging Details** 20 Tubes - Slant Tube 16 x 113 mm 12 months 1 box with 20 tubes, 16x113 mm glass tubes, ink 2-25°C labelled and metal cap. with: 6.5 ± 0.3 ml

Composition

Composition (g / 1600 ml):	
Potato starch	30.00
Asparagine	3.60
Magnesium citrate	0.60
Magnesium sulfate	0.24
Potassium dihydrogen phosphate	2.40
Malachite green	0.40
Glycerol	12 ml
Egg emulsion	1000 ml
Distilled water	600 ml

Description / Technique

Description

Löwenstein originally formulated a medium for cultivation of mycobacteria in which congo red and malachite green were incorporated for the partial inhibition of other bacteria. The present formula, developed by Jensen, differs in the citrate and phosphate content, does not contain congo red and has an increased malachite green concentration.

Lowenstein-Jensen Medium Base is a relatively simple formulation that requires supplementation in order to support the growth of mycobacteria. Glycerol (if required) and egg mixture are added prior to the inspissation process. These substances provide fatty acids and protein required for the metabolisms of mycobacteria. The coagulation of egg albumin during the sterilization provides a solid medium for inoculation purposes.

Technique

The sample must be treated according its origin and concentred if it is necessary. All the manipulations with the sample must be performed with the suitable safety standards. Inoculate the culture medium massively by spreading the sample in the surface. Use the glycerol-free culture medium when culturing glycerophobic mycobacteria. Incubate for four weeks at 35oC in horizontal position. After the hiding of the inoculum (2-3 days) the tubes are firmly tightened and aerated weekly. Typical colonial morphology requires a good oxygenation and absence of liquid in the surface. Check the tubes for colony growth after 10-14 days and then in weekly intervals. The final result is obtained after 8 weeks of incubation.

Appearance of colonies of Mycobacterium tuberculosis on Lowenstein-Jensen Medium with Glycerol or not.

Type humanus (R variant) - with glycerol Eugonic growth: Abundant, raised, crumbly, dry, usually yellowish (navel form) colonies -Glycerol-free The same pattern but with a poorly growth

Type bovinus (S variant) - with glycerol Sparse growth or no growth at all - Glycerol-free Dysgonic growth: flat, moist, glossy, confluent colonies (often nipple form) without pigment formation.

Type gallinaceous y Tipo poikilothermorum - with glycerol and Glycerol-free Rapid growth in the form of a moist, fairly abundant "lawn".

Precautions for use:

For professional use only.

Do not use if the product is contaminated, broken, or spoiled.

Store in a dark, dry place in their original packaging.

Avoid freezing and overheating.

The expiration date is the date of maximum inoculation.

The greenish color of the medium may change during its useful life. However should not be used when the medium has a intense blue color, by being exposed to sunlight.

Page 1 / 2 Revision date: 27/02/18





Reference: 4100

Technical Data Sheet

Product: LOWENSTEIN JENSEN MEDIUM

CEIVD

Quality control

Physical/Chemical control

Color: Light green pH: 6.8 ± 0.2 at 25°C

Microbiological control

Prepare a suspension from pure culture.

Loop spreading

Aerobiosis. Incubation inclined tubes for a maximum of 21 days

Microorganism	Growth
Mycobacterium gordonae ATCC® 14470	Good
Mycobacterium kansasii ATCC® 12478	Good
Mycobacterium tuberculosis ATCC® 25177	Good
Mycobacterium fortuitum ATCC® 6841	Good
Mycobacterium smegmatis ATCC® 14468	Good
Mycobacterium terrae ATCC® 15755	Good
Mycobacterium intracellulare ATCC® 13950	Good

Sterility Control

Incubation 7 days at 32.5 ± 2 °C and 7 days at 22.5 ± 2°C: - NO GROWTH

Bibliography

- · JENSEN, K.A. (1932) Reinzüchtung und Typenbestimmung von Tuberkelbazillenstammen. Zbl. Bakt. I. Orig. 125:222-239
- · LOWENSTEIN, E. (1931) Die Züchtung der Tuberkelbazillen aus dem strömenden Blute. Zbl. Bakt. I. Orig. 120:127-129
- · BALOWS A., W.J. HAUSLER JR, K.L. HERRMANN, H.D. ISENBERG, H. JEAN · SHADOMY (1991) Manual of Clinical Microbiology 5th ed ASM Press, Washington DC.
- · PFYFFER G.E., B.A. BROWN-ELLIOT & R.J. WALLACE Jr. (2003) Mycobacterium: General Characteristics, Isolation, and Staining Procedures in Manual of Clinical Microbiology 8th ed. by Murray, Baron, Jorgensen, Pfaller and Yolken. ASM Press, Washington DC.

Page 2 / 2 Revision date: 27/02/18