75/387027/1011

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75/387027/1011

Aqua B. Braun

Irrigation solution, sterile and endotoxin-free

Composition
1000 ml solution contain:
Water for Injection 1000 ml

Indications
For irrigation and cleansing during surgical intervention

For irrigation of wounds and burns

For moistening wound tamponades, cloths and dressings
For rinsing and cleansing of bladder catheters
For irrigation and cleansing in stoma care

For rinsing and cleansing of instruments and equipment
For filling respiratory air humidifiers (the use of container equal or higher than 100 ml is recommended)

For mechanical irrigation of the eyes (the use of 30 ml container is recommended)

Dosage
The amount of fluid depends on the clinical condition.

Overdosage
Overdosage during wound irrigation and intra-operative application can lead to absorption of irrigation fluid, that can lead to hypervolaemia in the form of hypotonic

hyperhydration (hyponatraemia). Headache, nausea, restlessness and disorientation are symptoms

of hyperhydration. In severe cases coma-like states can occur.
Countermeasures in cases of overdosage
Discontinuation of irrigation, correction of the water and electrolyte balance depending on the particular clinical situation.

Warnings and precautions
Do not use for infusion.

Only use the solution if the seal is undamaged and the solution is clear.
Opened containers must not be saved, quantities left over must be discarded.

Re-use of single-use devices creates a potential risk of patient or user. It may lead to contamination and/or impairment of functional capability. Contamination and/or limited functionality of the device may lead to injury, illness or death of the patient.

Store out of reach of children.
Aqua B. Braun must not be used for intratecal irrigation procedures.

Adverse side effects
None are known.

Expiry date
The solution must not be used after the expiry date given on the label.

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