



## H-Pette® Dispenser With Blade

*For blood smear preparation*


*The safe, time-saving, all-in-one dispenser*

- Easy delivery of a blood sample to a glass slide
- Reduces the risk of lab-acquired infections by minimizing aerosols and exposure to blood borne pathogens
- No special training, no special fittings, or inserts required



[www.labcon.com](http://www.labcon.com)



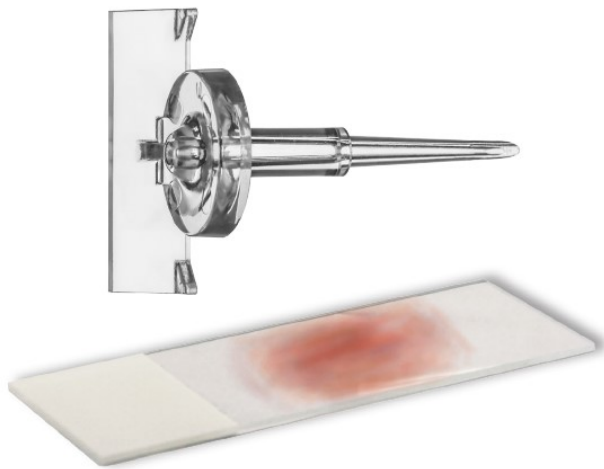

## H-Pette® Dispenser with Blade

Labcon's H-Pette® Dispenser makes blood slide preparation safe and easy. Simply insert the H-Pette® directly into the stopper of any size blood collection tube, dispense one drop onto a glass slide, and use the blade to smear the drop across the slide. Since you don't have to open the primary sample tube, you greatly reduce the risk of lab-acquired infection by minimizing aerosols and exposure to blood borne pathogens.

H-Pette® simplifies blood smear preparation by cutting out several steps, thereby improving efficiency, saving time, and reducing wasted materials and blood samples. Using the H-Pette eliminates the need for sticks, capillaries, or syringes to dispense blood on the slide. No special training, no special fittings or inserts required.

Compliant with the IVD Directive 98/79/EC and lot tested Endotoxin (Pyrogen) Free.



### Using the H-Pette®




**1**  
**Insert** H-Pette's cannula into the center of the stopper of the collection tube. Gently press until the cannula completely penetrates the septum.\*

\*The cannula is to be inserted in existing puncture in the stopper of the collection tube.



**2**  
**Invert** collection tube with H-Pette's round edge pointed down, as shown. Gently press round edge to the glass slide to dispense one blood drop onto the slide.

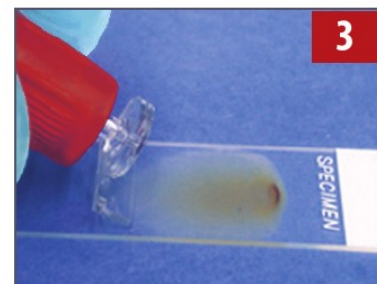

**3**  
**Rotate** tube at an angle with H-Pette's blade pointed down, as shown. Use the blade to make a smear just as you would a glass slide.




Labcon North America  
Petaluma, CA, U.S.A

+1.707.766.2100  
info@labcon.com  
www.labcon.com



### H-Pette® Ordering Information

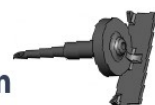

Catalog #	Description	Total H-Pettes®
7620-540-000-9	1 Boxes of 100 H-Pettes®	100
7620-500-010-9	5 Boxes of 100 H-Pettes®	500

H-Pette™ is made of polystyrene. Cannula tip length is 0.18".  
Fits all standard 13mm and 16mm blood collection tubes.

Product details are current when published; subject to change without notice. Labcon, the Labcon logo, and variations on the foregoing are registered (®) or unregistered trademarks and service marks, of Labcon North America. © Copyright 2017 Labcon. All rights reserved.

REV 1...10.10.17