





C-Pette® and C-Pette® E Blood Transfer Dispenser

- For safe and efficient blood dispensing
- Ideal for Point-Of-Care (POC) use with any analyzer
- Minimizes aerosols
- Reduces the risk of lab-acquired infections



www.labcon.com



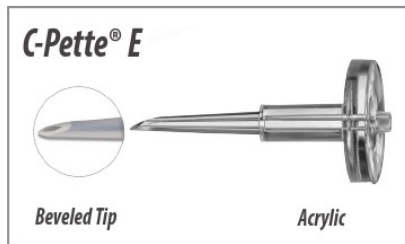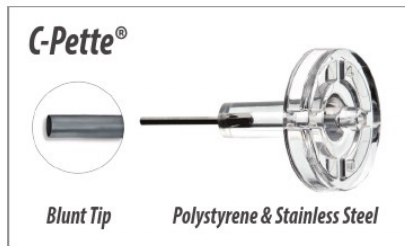

Labcon C-Pette® & C-Pette® E

Labcon's C-Pette and C-Pette E makes delivering one precise drop of sample into test cards and point of care cassettes simple and efficient. Using these pettes improves clinician and patient safety. Since you don't have to open the primary sample tube, you greatly reduce the risk of lab-acquired infection by minimizing aerosols and exposure to blood borne pathogens.

Labcon's C-Pette has a narrow stainless steel cannula that is designed for low puncture force. The C-Pette E is made of durable acrylic and is the economical alternative to the C-Pette. Both pettes fit all standard blood collection tubes and are ideal for use with many point of care analyzers such as the Abbott® i-Stat® Helena® Abrazo® and other POC Analyzers that requires the delivery of a sample to a test cartridge or cassette.

Labcon's pettes are all compliant with the IVD Directive 98/79/EC and are lot tested Endotoxin (Pyrogen) Free.



i-STAT is a registered trademark of the Abbott Group of Companies

Using the C-Pette® & C-Pette® E

Step 1. Grasp the primary blood collection tube by the plastic skirt covering the rubber stopper and insert the metal cannula thru the stopper.

Step 2. Re-suspend cells just prior to dispensing by holding the tube by the plastic skirt and inverting gently as needed.

Step 3. Place the C-Pette dropper tip into the cassette loading port.

Step 4. Push down and release to completely fill cassette reservoir. If a larger sample volume is required, repeat push and release of tube.



Labcon North America
Petaluma, CA, U.S.A

+1.707.766.2100
info@labcon.com
www.labcon.com



C-Pette® Ordering Information

Item #	Description	Total
7630-500	5 boxes/100	500
7630-540	1 box/100	100



C-Pette® E Ordering Information

Item #	Description	Total
7625-500	5 boxes/100	500
7625-540	1 box/100	100

Product details are current when published; subject to change without notice. Labcon, the Labcon logo, and variations on the foregoing are registered (®) or unregistered trademarks and service marks, of Labcon North America. © Copyright 2017 Labcon. All rights reserved.

REV 0_ 4.11.17