

U-Pette[™] Blood Dispenser

For safe and efficient blood dispensing into test cards and point of care cassettes



www.labcon.com



1 sur 2 25/05/2023, 10:29





Labcon U-Pette™

Labcon's U-Pette[™] provides a quick and easy way to dispense blood into a test cassette without removing the cap from the collection tube. This greatly reduces the risk of infection due to aerosol exposure. Made of acrylic and stainless steel, U-Pette[™]'s flexible sides allow for efficient and precise delivery of required sample amount, whether it's one droplet or continuous flow dispensing. Labcon's U-Pette[™] fits all standard blood collection tubes and is ideal for use with many point of care analyzers such as the Abbott® i-Stat® Helena® Abrazo® and other POC Analyzers that requires the delivery of a sample to a test cartridge or cassette.

Compliant with the IVD Directive 98/79/EC and lot tested Endotoxin (Pyrogen) Free.











i-STAT is a registered trademark of the Abbott Group of Companies

Using the U-Pette™



Grasp center of the U-Pette™ at the side of the disc as shown.



Insert metal cannula through existing needle puncture of the capped tube.



Invert tube and place tip all the way into sample well until it reaches the bottom.



Use tabs to counter pressure and push down on tube to dispense sample as needed.



Labcon North America Petaluma, CA, U.S.A

+1.707.766.2100 info@labcon.com www.labcon.com







	ordering information	
Catalog #	Description	Total U-Pettes™
7626-500	5 Boxes of 100 U-Pettes™	500
7626-540	1 box of 100 U-Pettes™	100

U-Pette™ is made of acrylic with a stainless steel cannula. Cannula tip length is 0.18". Fits all standard 13mm and 16mm blood collection tubes.

Product details are current when published; subject to change without notice. Labcon, the Labcon logo, and variations on the foregoing are registered (*) or unregistered trademarks and service marks, of Labcon North America. © Copyright 2017 Labcon. All rights reserved.

EV 1_6.14.17