TST Single Use Bowie Dick Test Pack - Instructions For Use

A steam penetration test for daily use in pre-vacuum **STEAM**[ⓘ] sterilizers working at 134°C to 137°C for up to 3.5 minutes as required in EN ISO 17665-1

Conforms to: ISO 11140-4 Type 2

Instructions for Use

- Pre-heat the sterilizer by running one complete cycle.
- Record relevant test cycle information on the outside of the pack in the space provided (if required).
- Place the unopened TST Bowie Dick Test Pack horizontally between 100mm and 200mm over the chamber drain or as near as is practically possible on a perforated support, in a preheated empty sterilizer.
- Run a cycle of the machine, using the same air removal stage as that to be used for sterile production, at 134°C – 137°C for up to 3.5 minutes. The pack should be immediately removed from the sterilizer and the results checked. Care should be taken, as the TST pack may be warm. Exposure to 134°C – 137°C for greater than 3.5 minutes invalidates the test. **Note:** The indicator sheet is located in the centre of the pack.
- If the sterilizer air removal stage is functioning correctly and enabling rapid and even steam penetration, the ink pattern on the TST indicator sheet will have changed from yellow to dark blue / purple.
- If even and rapid steam penetration did not occur, the ink pattern on the test sheet will show a yellow or brown colour towards the centre of the pack. Refer to Colour Guide right.
- The information should be recorded on the indicator sheet, which should then be filed in a cool, dry, dark place away from direct light.

Interfering substances or conditions known to affect the performance of the indicator
After a period of 24 – 48 hours, the blue/purple of the test sheet may darken. Refer to Colour Guide right. This is completely normal and is due to the test sheet drying out after processing.

Also after a similar period, the blue/purple of the indicator sheet may assume a grey colour. This is due to over-processing i.e. long heating up times and/or temperature / times exceeding the stated working conditions of the pack, which may invalidate the test.

If the processed indicator sheets are exposed to strong light, some fading of colour may occur.

Effect of Steam Quality

Steam used for sterilization should have a dryness factor of 0.90 to 1.00. Steam with a dryness factor at the lower limit or below may cause the TST indicator sheet to have a mottled appearance towards the centre of the sheet. Although this does not signify a failure of the Bowie Dick test (unless accompanied by yellow or brown discolouration in the centre of the indicator sheet), it does indicate the existence of unsatisfactory wet steam, and the problem should be addressed.

Steam with a dryness of factor of 1.00 or above may contribute to superheating of the steam within the sterilizer chamber. If this superheat is excessive, the TST pack (and any other load) may be exposed to temperatures exceeding 150°C. If this does occur, the TST indicator sheet may have a smudge or blurred appearance around the edges. Although this does not signify a failure of the Bowie Dick test (unless accompanied by a yellow or brown discolouration in the centre of the indicator sheet), it does indicate the existence of unsatisfactory superheat conditions, and the problem should be addressed.

TST Single Use Bowie Dick Test Pack - Instructions For Use

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The company, whose policy is one of continuous improvement, reserves the right to make, without prior notification, changes in materials, dimensions and designs which, having regard to all circumstances, it thinks reasonable or desirable provided that the customer shall not be required to accept goods substantially different from those which are specified within the order.

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BROWNE
<p> Sterility Assurance Products World Class and Globally Trusted</p>
<p> STERIS</p>

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